

We invite you to watch the 2018 COSM Checkpoint symposium as Dr. Salvatore Caruana, Director of the Division of Head and Neck Surgery, Columbia University Medical Center, and Dr. Raymond Chai, Assistant Professor, Otolaryngology, Mount Sinai Medical Center, New York, discuss the use of our Checkpoint nerve stimulation technology as an alternative to conventional nerve monitoring. Worth noting is their somewhat different perspectives on how intraoperative nerve stimulation with Checkpoint is helpful as an alternative or adjunct to conventional nerve monitors.




This symposium discusses the use of Checkpoint intraoperative nerve stimulation technology for nerve protection and assessment in head and neck procedures. Dr. Caruana and Dr. Chai review their use of Checkpoint's stimulation technology in Parotidectomy, Neck Dissection and Thyroidectomy procedures, including video of case examples, and take a variety of questions from the audience.

The full video can be viewed by clicking [here](#) or by clicking on the picture above.

Next, we'll discuss the use of the Checkpoint® Head & Neck Stimulator/Locator in Thyroidectomy procedures. While everyone would agree that visualization of the Recurrent Laryngeal Nerve (RLN) is the "Gold Standard" in Thyroidectomy surgery, visual identification and preservation of the RLN alone does not guarantee normal postoperative vocal cord function. Anatomical continuity of a motor neuron does not ensure function because there are other causes of nerve injuries such as stretching and excessive dissection of the nerve with ischemia. Additionally, there is ongoing debate on the benefit of the use of Intraoperative Nerve Monitoring (IONM) as a tool to reduce the risk of nerve injury and improve outcomes. IONM has proven to be highly useful in certain circumstances but has not been definitively proven to protect the nerve any more effectively than the gold standard of nerve visualization. The bottom line is that the key to good surgical outcomes, is first and foremost, good surgical skills. We have to remember that any technology, and in our case, "new technology" should be considered an added "tool" to augment or provide additional information to the surgeon during surgical procedures.

Nofal and El-Anwardoi remind us that routine identification of the recurrent laryngeal nerve


during thyroidectomy represents the “standard of care” in thyroid surgery as it reduces the incidence of definitive postoperative vocal cord palsy. Intraoperative laryngeal palpation during RLN stimulation is a safe, reliable method for neural monitoring that can assist in RLN identification and assessment during thyroid and parathyroid surgery. Unfortunately, as the NIM and other IONM devices have been introduced into head & neck surgery, many residents and surgeons have either forgotten or were never trained in this procedure and only rely on IONM to ensure nerve integrity. Regardless of surgical approach, technique and advancement of technology, a safe and reliable means of RLN localization and verification can sometimes be a clinical challenge, especially when the thyroid gland is large and the surrounding soft tissue planes are distorted. Checkpoint offers the surgeon a new, safe reliable technology to assist in this localization and verification.

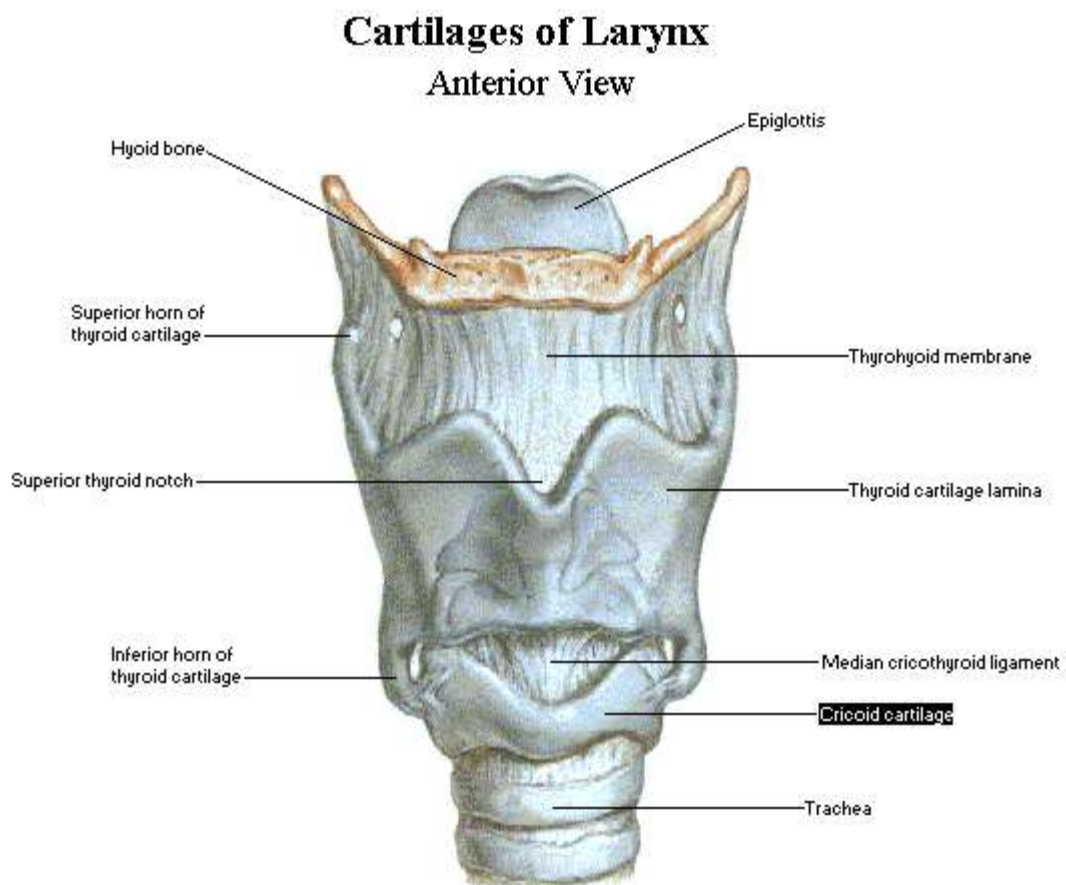

RLN Testing

Simply locate the Cricoid Cartilage with your finger tip and slide finger posterior placing your fingertip directly on the posterior and/or lateral cricoarytenoid muscles. Utilizing Checkpoint’s biphasic waveform and ability to produce a tetanic muscle contraction, stimulate the RLN at 2mA which produces a robust contraction, which is easily palpable and reproducible.

Intrinsic Muscles of Larynx Lateral Dissection

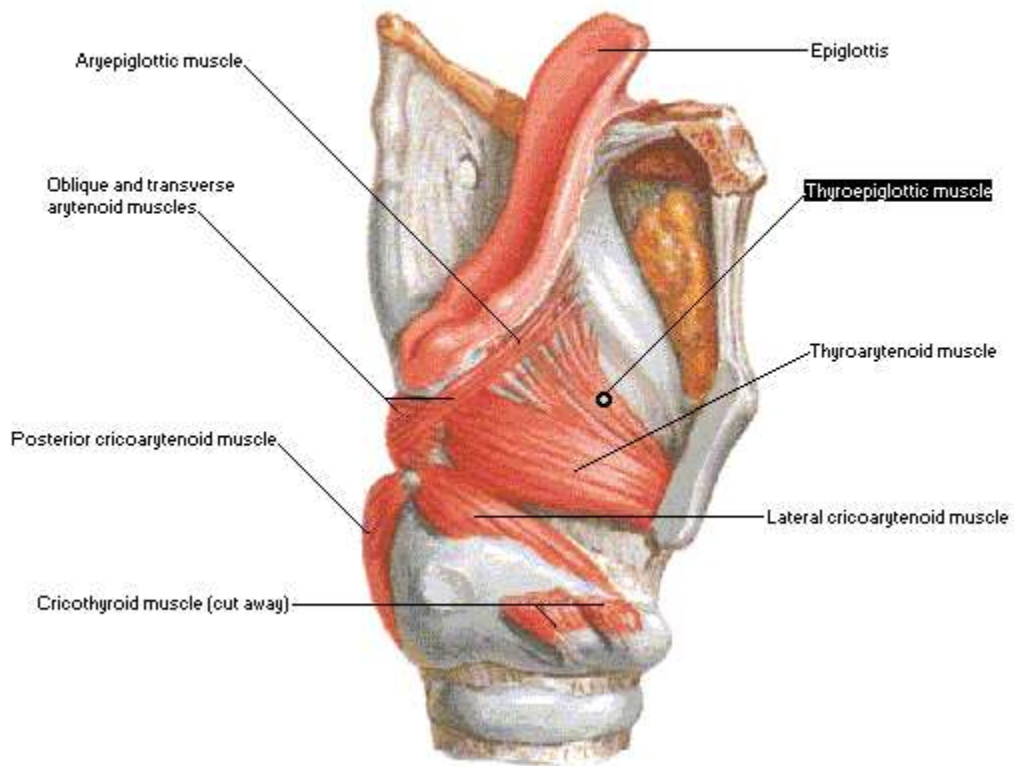

Who Might Consider Checkpoint in Thyroid Surgery?

- Surgeons who primarily want to safely and reliably verify function of the RLN during or after the removal of the Thyroid and before moving to the second side
- Surgeons who only use IONM for RLN confirmation
- Surgeons who currently use IONM, but are looking for a more cost effective, safe alternative
- Surgeons who do not use IONM, but want to reliably test RLN

Contact Checkpoint at info@checkpointsurgical.com if interested in scheduling a trial.